SURGIFOLD™

Manifolds and Stopcocks
Collecteurs et Vannes de Régulation
Coletores e Válvulas Reguladoras
Colectores y Válvulas Reguladora

Table of Contents

ENGLISH (en)	2
FRENCH (fr)	3
PORTUGUESE (pt)	4
SPANISH (es)	6

Instructions for Use

Description:

ALSPL manifolds and stopcocks are disposable intervention accessories which provide multiple injector ports for fluid administration. The handles of the manifolds and stopcocks are marked with arrows and 'ON / OFF' indicating the direction of flow for ease of the user. All injector ports have luer lock fittings for easy use and compatibility.

Indications for Use:

ALSPL's manifolds and stopcocks are intended to facilitate and control fluid flow between tubing, catheters and any other devices during diagnostic and interventional procedures.

Contraindications:

- No specific contraindications known
- As manifolds and stopcocks are only an accessory for procedures, the contraindications are limited to the contraindications for the respective procedures

Warnings and Precautions:

1. Please read all packaging insert warnings, precautions, and instructions before use. Failure to do so may result in severe patient injury or death.
2. Warning: Sterile by EO, Single Use: Do not reuse, reprocess or re-sterilize under any circumstance. Blood and other foreign material can collect in the inner parts of the product which cannot be removed.
3. Warning: Do not use if the packaging is damaged or opened unintentionally before use. This will lead to loss of sterility.
4. Warning: Rx Only. The products must be used by trained physicians
5. Precaution: Aseptic techniques must be used wherever possible, especially during removal from packaging.
6. Precaution: Ensure the connection is secure as insecure connections can lead to air in the system which can cause an embolism which in rare circumstances leads to death.
7. Precaution: The connection should be hand tightened, but not over tightened as that can lead to cracks which may cause an embolism
8. Precaution: Check products carefully to ensure there is no entrapped air which can lead to an embolism
9. Precaution: Connections should only be made with devices compliant to ISO 80369-7

Instructions for Use:

1. Open the manifold or stopcock aseptically from the packaging
2. Connect with the required devices
3. Ensure connections are secure and hand tightened. Avoid over tightening as it can lead to cracking
4. The handles are marked with arrows and 'ON/OFF' markings for ease of use

Disposal:

5. Manifolds and stopcocks can be a biological hazard if disposed incorrectly. They should be disposed using recognised medical procedures and industry best practices. Local laws and regulations should be taken into account during disposal

Potential Complications / Adverse Effects

Manifolds and stopcocks are accessories used in various medical procedures. The potential complications or adverse effects caused during use are limited to the procedure being performed.

Warranty and Limitations

Manifolds and stopcocks are sold in 'as is' condition. The entire risk as to the quality and performance of the product is with the buyer. ALSPL disclaims all warranties, expressed or implied,

with respect to its products, including but not limited to, any implied warranty of merchantability or fitness for a particular purpose. ALSPL shall not be liable to any person for any medical expenses or any direct or consequential damages resulting from the use of any product or caused by any defect, failure, or malfunction of any product, whether a claim for such damages is based upon warranty, contract, tort, or otherwise. No person has any authority to bind ALSPL to any representation or warranty with respect to its products.

Notice: Any serious incident which occurs in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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