QUICK™
Introducer Needle
Aiguille de Ponction
Agulha de Punção
Aguja de Punción

Table of Contents

ENGLISH (en)	2
FRENCH (fr)	3
PORTUGUESE (pt)	4
SPANISH (es)	5

Instructions for Use

Description:

ALSPL Introducer Needles are disposable intervention accessories which provide access path into the vasculature. They are composed of a stainless-steel cannula and a plastic hub.

Indications for Use:

ALSPL's Introducer Needles are intended to facilitate percutaneous introduction of guidewires.

Contraindications:

- No specific contraindications known
- As Introducer Needles are only an accessory for procedures, the contraindications are limited to the contraindications for the respective procedures

Warnings and Precautions:

1. Please read all packaging insert warnings, precautions, and instructions before use. Failure to do so may result in loss of blood and delay to the procedure.
2. Warning: Sterile by EO, Single Use: Do not reuse, reprocess or re-sterilize under any circumstance. Blood and other foreign material can collect in the inner parts of the product which cannot be removed.
3. Warning: Do not use if the packaging is damaged or opened unintentionally before use. This will lead to loss of sterility.
4. Warning: Rx Only. The products must be used by trained physicians
5. Precaution: Aseptic techniques must be used wherever possible, especially during removal from packaging.

Instructions for Use:

1. Stretch skin and hold tightly
2. Ensure bevel of needle is facing upwards and puncture and insert the needle in the required vessel
3. Aspirate to ensure proper placement
4. Introduce a guidewire as per the procedure being performed
5. Remove needle once wire is in place

Disposal:

6. Needles can be a biological and physical hazard if disposed incorrectly. They should be disposed using recognised medical procedures and industry best practices on contaminated sharps disposal. Local laws and regulations should be taken into account during disposal

Potential Complications / Adverse Effects

Introducer Needles are accessories used in various medical procedures. The potential complications or adverse effects caused during use are limited to the procedure being performed.

Warranty and Limitations

Introducer Needles are sold in 'as is' condition. The entire risk as to the quality and performance of the product is with the buyer. ALSPL disclaims all warranties, expressed or implied, with respect to its products, including but not limited to, any implied warranty of merchantability or fitness for a particular purpose. ALSPL shall not be liable to any person for any medical expenses or any direct or consequential damages resulting from the use of any product or caused by any defect, failure, or malfunction of any product, whether a claim for such damages is based upon warranty, contract, tort, or otherwise. No person has any authority to bind ALSPL to any representation or warranty with respect to its products.

Notice: Any serious incident which occurs in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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