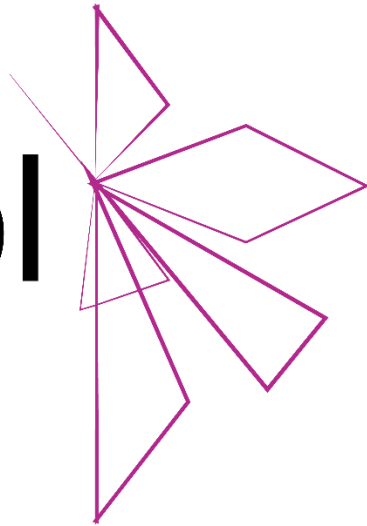
alspl

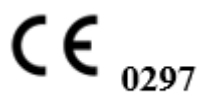
AQUA™

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Instructions for Use:**Description:**

The control syringe is a single use sterile syringe with a male luer lock at its end for easy connection with various cardiac catheters such as angiography catheters. It is one of the primary elements of any angiography kit as it used throughout an angiographic procedure for smooth injections of saline or contrast. The syringe is marked at each millilitre for ease of the user.

Indications for Use:

ALSPL's control syringes are intended to facilitate hand injections of contrast medium, saline or any other liquid during angiography or other radiology procedures.

Contraindications:

- No specific contraindications known
- As the control syringe is only an accessory for procedures, the contraindications are limited to the contraindications for the respective procedures

Warnings and Precautions:

1. Please read all packaging insert warnings, precautions, and instructions before use. Failure to do so may result in severe patient injury or death.
2. Warning: Sterile by EO, Single Use: Do not reuse, reprocess or re-sterilize under any circumstance. Blood and other foreign material can collect in the inner parts of the syringe which cannot be removed; therefore, syringes are for single use only.
3. Warning: Do not use if the packaging is damaged or opened unintentionally before use. This will lead to loss of sterility.
4. Warning: The syringe is not intended for balloon catheter inflation as it does not have a pressure gauge and can result in over inflation and rupture of the balloon
5. Warning: Rx Only. The control syringe must be used by trained physicians
6. Precaution: Aseptic techniques must be used wherever possible, especially during removal from packaging.
7. Precaution: Ensure the connection is secure as insecure connections can lead to air in the system which can cause an embolism which in rare circumstances leads to death.
8. Precaution: The connection should be hand tightened, but not over tightened as that can lead to cracks which may cause an embolism
9. Precaution: Check syringe carefully to ensure there is no entrapped air which can lead to an embolism
10. Precaution: Connections should only be made with devices compliant to ISO 80369-7

Instructions for Use:

1. Open syringe aseptically from the packaging
2. Fill with the required liquid and ensure there are no air bubbles present
3. Connect with the catheter ensuring no air has entered the system
4. Inject as required
5. Disconnect by rotating the luer lock

Disposal:

6. Control Syringes can be a biological hazard if disposed incorrectly. They should be disposed using recognised medical procedures and industry best practices. Local laws and regulations should be taken into account during disposal

Potential Complications / Adverse Effects

Control Syringes are accessories used in various medical procedures. The potential complications or adverse effects caused during use are limited to the procedure being performed.

Warranty and Limitations

Control Syringes are sold in 'as is' condition. The entire risk as to the quality and performance of the product is with the buyer. ALSPL disclaims all warranties, expressed or implied, with respect to its products, including but not limited to, any implied warranty of merchantability or fitness for a particular purpose. ALSPL shall not be liable to any person for any medical expenses or any direct or consequential damages resulting from the use of any product or caused by any defect, failure, or malfunction of any product, whether a claim for such damages is based upon warranty, contract, tort, or otherwise. No person has any authority to bind ALSPL to any representation or warranty with respect to its products.

Notice: Any serious incident which occurs in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

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